NO

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GB

**First Aid advice - Main rules for First Aid****Make certain the patient:**

- a) has unblocked airways
- b) is breathing
- c) has proper circulation

**When the patient is unconscious, but breathing:**

Put the patient in the recovery position

**When the patient is unconscious and not breathing:**

Start Pulmonary and Cardiac Resuscitation (CPR). Blow the air in slowly, twice, followed by 15 chest compressions  
Get medical assistance

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**Bleeding:**

Stop the bleeding. Cover the wound with dressing or adhesive bandage  
Get medical assistance

**Injuries by fire:**

Start cooling down the burned area with water or a burn-bandage as soon as possible  
Open the pack, take out the compress and place it on the burn. Do not remove burnt-in remains from the wound; do not cut open blisters or touch injured/damaged tissue  
Get medical assistance

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**GB**

Items	Product	Usage
1	Stretch gauze bandage, 8 x 400 cm	General purpose fixation
12	Plasterstrips, single, 2,5 x 7,5 cm	Cuts and small wounds
2	Gauze dressing, 10 x 10 cm pack of 2	Place on wound
1	Triangular bandage	Immobilisation
1	Mouth-to-mouth mask	Mouth-to-mouth resuscitation
2	Disposable gloves	Hygienic protection
2	Wound cleansing napkins	Cleaning wound and skin
1	Scissors	Cutting bandages and clothes
1	Contents/ First Aid guide	

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