NO

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GB

First Aid advice - Main rules for First Aid**Make certain the patient:**

- a) has unblocked airways
- b) is breathing
- c) has proper circulation

When the patient is unconscious, but breathing:

Put the patient in the recovery position

When the patient is unconscious and not breathing:

Start Pulmonary and Cardiac Resuscitation (CPR). Blow the air in slowly, twice, followed by 15 chest compressions
Get medical assistance

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Bleeding:

Stop the bleeding. Cover the wound with dressing or adhesive bandage
Get medical assistance

Injuries by fire:

Start cooling down the burned area with water or a burn-bandage as soon as possible
Open the pack, take out the compress and place it on the burn. Do not remove burnt-in remains from the wound; do not cut open blisters or touch injured/damaged tissue
Get medical assistance

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GB

Items	Product	Usage
4	Gauze dressing, 10 x 10 cm pack of 2	Place on wound
2	First Aid dressing, 12 cm	Bleeding and wounds
2	Stretch gauze bandage, 6 x 400 cm	General purpose fixation
2	Stretch gauze bandage, 8 x 400 cm	General purpose fixation
1	Triangular bandage	Immobilisation
1	Mouth-to-mouth mask	Mouth-to-mouth resuscitation
6	Disposable gloves	Hygienic protection
12	Plasterstrips, single, 2,5 x 7,5 cm	Cuts and small wounds
1	Plaster, 6 x 10 cm, packet of 5 pcs.	Cuts and small wounds
1	Nonwoven tape, 1,25 cm	Fixation of wound dressing
1	Scissors	Cutting bandages and clothes
10	Wound cleansing napkins	Cleaning wound and skin
1	Reflex bandoler	Be visible
1	First Aid guide	Guidance on first aid
1	Contents	

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